

Instructions for Use

IMPORTANT

- Do not attempt to use the Face Shield before completely reading and understanding the information contained in this Instructions for Use.

Device Description

The Protostudios Face Shield is a face shield to be worn by the user to protect from transfer of microorganisms, body fluids, and particulate material. The Protostudios Face Shield is composed of four parts: the clear plastic guard (polyester), halo head piece made of polypropylene, and foam. The Protostudios Face Shield can be cleaned and reused.



Contents of Packaging

- One Protostudios Face Shield
- One instructions for use

How Supplied

Devices are bulk packaged fully assembled, clean and nonsterile.

Indications for Use

The Protostudios Face Shield is intended to be worn on the user's head to protect the user from transfer of microorganisms, body fluids, and particulate material.

Contraindications

The Protostudios Face Shield should not be used on any user for who:

- Has abnormal cranial anatomy
- The halo head piece does not form a secure fit

Warnings

- Do not sterilize.
- Improper use of this device may cause injury.

Precautions

- If there are any variations between these Instructions for Use and either your facility's policies, those variations should be brought to the attention of the appropriate responsible hospital personnel for resolution before proceeding with device.
- Prior to use, inspect devices to ensure proper function and condition.
- Do not use the Protostudios Face Shield if it does not satisfactorily perform its intended function or if it has physical damage.
- Do not use Protostudios Face Shield if package is open or damaged.
- Avoid mechanical shock or overstressing the device.

Directions for Use

Protostudios Face Shield Use Procedure

1. To put on the Protostudios Face Shield first place the hooks toward the back of the head, and pull the clear guard down until the foam side of the halo head piece is resting against the forehead, and the clear guard is covering the eyes, nose, and mouth. The motion should imitate putting on a hat.
2. Optional: If the Protostudios Face Shield does not form a secure fit a rubber band may be added to the hooks on the back of the head.
3. After use remove the halo head piece from forehead.
4. Dispose of the Protostudios Face Shield in biohazard waste if available. If not available, wipe down with Isopropyl before disposing in trash.

Instrument Cleaning and Sterilization

Protostudios Face Shield is provided clean, non-sterile, and ready for use. Isopropyl alcohol may be used to clean the clear plastic shield at least daily if being reused.

MADIE

 THE UNIVERSITY OF IOWA



MADE, The University of Iowa
25 S. Grand Avenue
Iowa City, IA 52242
(319) 384-3425

For product ordering or reporting of adverse events, please call (319) 384-3425, email iowa-made@uiowa.edu, or visit iowamade.org

Rev. C (05/22/20)
ID0005